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Mya[®]

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Do you feel prisoner
of the Cellulite ?

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LIPO-MODELLING
with medium frequencies
and ultrasounds

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Mya® can perform a **Bio Impedance analysis** of the body, by means the positioning of special electrodes on hand and foot of the client. A sinusoidal signal passes through the body in a total painless way. It is sufficient to set a data series relating to body structure and by means of proper algorithms, the equipment will process following data: **free fat mass, fat mass, total water, intra-cell water and total advised sessions, which will be divided into four groups: slimming, draining, cellulite*, firming-up treatments.** It is possible to repeat the **BIA test**, comparing the results with previous ones, in order to analyze the variation and to fix a maintenance cycle treatments.

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The number of advised treatments can be modified according to client needs, before being stored on the personal chip card. For each menu of the advised treatments [cellulite*, slimming, draining, toning] **Mya®** will suggest the right program among the 22 methods dedicated to the beauty defect. At the end of the treatment the chip card will count down the performed session. Chip card can be used for new clients, by deleting old data contained, before storing data of the new client.

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Simply Unique For slimming, re-modelling and anti-cellulite* treatments, which grant immediate and long-lasting results.

With Mya you can:

- Make the Bio Impedance Test to set personalized protocols for every client.
- Choose the most suitable treatment for the customer among the 11 programs which perform the contemporary emission of electro stimulation and ultrasound, by means of 16 emitting plates.
- Choose the most suitable treatment for the client, between the 11 programs which perform only electro-stimulation by means of pre-gelled electrodes, combining them if needed with plates which will emit only ultrasounds.
- Store the client data on the Chip Card, in order to set personalized protocols.

Toning, slimming, lipolysis, modelling, relaxing massage, unsightliness of cellulite, drainage

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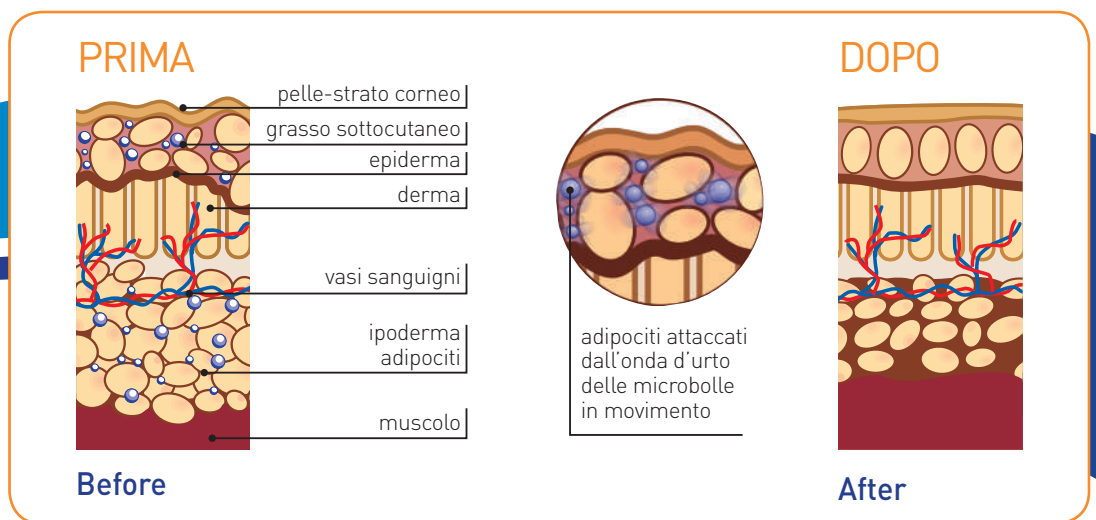
Method of contemporary emission of electro-stimulation and ultrasounds, by means of 16 emitting plates.

What makes **Mya**[®] unique is the patented system by Cesare Quaranta Srl which includes the combined **emission of two separate sources, electro-stimulation and ultrasound on a single support**, 16 plates, which can emit the two sources at the same time. 11 programs between which it is possible **to choose the ideal treatment for slimming, cellulite*, drainage, lipolysis, firming up, tissue and muscle toning.** The ultrasound weakens cellulite* tissue structure, mainly fat cells membrane; the micro-massage made at tissue level, the local heat production and the essential **draining effect induce a progressive unfastening of the cellulite* nodule.**

Method of electro-stimulation by means of electrodes and plates emitting only ultrasounds:

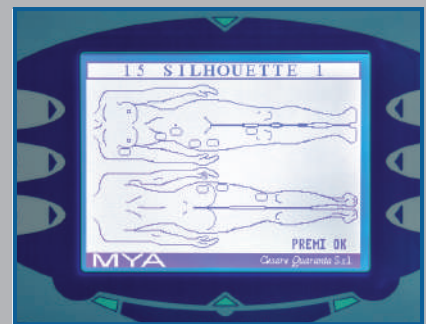
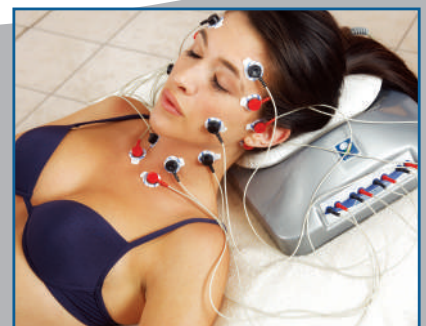
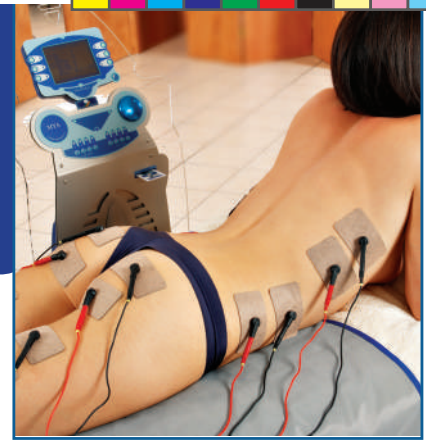
further 11 programs working by means of 8 channel for electro-stimulation which allow the positioning of 32 electrodes of different dimensions, according to the area to be treated and to the chosen program. With **Mya**[®] it is also possible **to combine the plates, which will emit only ultrasounds, to the electro-stimulation with electrodes.** The combining of these distinct methods allows **maintain the results obtained and to treat-quickly and effectively- different beauty defects on different body areas.**

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- **4 devices in one** to optimize the results offered to clients.
- **22 Programs** with the possibility of personalize protocols for a better client/equipment managing.
- **A big variety of wave-forms** and frequencies oriented to the best final results.
- **8 channels which can be doubled into 16**, allowing the usage of **32 electrodes**.
- **Pre-set programs which can be easily modified.**
- **Viewing of body diagrams**, upon selection of the desired program, in order to allow the correct positioning of the electrodes and plates and the maximum quickness while preparing phase.
- **Self-test to check** the correct functioning of the equipment.
- **A handy bumbag** to place 16 plates and cables for electro-stimulation and all consumption material
- Welcome vocal message.
- **Refined design, awarded with Aesthetic Award**, which makes Mya® an elegant a unctional furnishings element; with its vertical and compact shape, it is easy to be moved in absolute security thanks to its wheels with break system.
- An **encoder** allows to easily and quickly select menus and to set parameters of different programs.
- **A standard supply of reliable and high-quality accessories.**

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